



SECUR™

has you covered







Made to roll comfortably onto a finger or toe right out of the package.

PolyMem Finger/Toe dressings help reduce edema, bruising, pain and inflammation when applied to open or closed injuries. When applied to open wounds the dressings continuously cleanse, fill, absorb and moisten wounds. The dressings help relieve both persistent and procedure related pain associated with the injury throughout the healing process.

THE IDEAL CHOICE FOR MANAGING:

- Sprains
- Strains
- Contusions
- Abrasions
- Lacerations
- Burns
- Ulcers
- Matricectomies




Ref	Description	Dimensions	Packaging
 4401	#1 Finger/Toe Dressing	Ring Size 4 - 8; UK H - Q; EU 6.68 - 17.15	6 Per Box 5 Boxes Per Case
 4402	#2 Finger/Toe Dressing	Ring Size 8 - 12; UK Q - Y; EU 17.15 - 27.51	6 Per Box 5 Box Per Case
 4403	#3 Finger/Toe Dressing	Ring Size 12 - 16; UK Y - Z+7; EU 27.51 - 38.00	6 Per Box 5 Box Per Case
 1401	#1 Silver Finger/Toe Dressing	Ring Size 4 - 8; UK H - Q; EU 6.68 - 17.15	6 Per Box 5 Boxes Per Case
 1402	#2 Silver Finger/Toe Dressing	Ring Size 8 - 12; UK Q - Y; EU 17.15 - 27.51	6 Per Box 5 Box Per Case
 1403	#3 Silver Finger/Toe Dressing	Ring Size 12 - 16; UK Y - Z+7; EU 27.51 - 38.00	6 Per Box 5 Box Per Case


PolyMem Wound Care Dressings Are Latex Free



1 Measure to determine length of dressing needed, cut off excess.

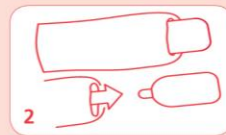

2 Remove the insert from the rolled end and discard.

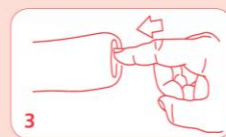

3 Insert the finger into the rolled end of the dressing.



4 Push the finger into the dressing and begin rolling.



5 Roll the dressing on the finger.



6 The dressing should fit securely on finger or toe.

This is an overview. Please see package insert for complete instructions.





SECUR™

has you covered

PolyMem®

PolyMem Finger/Toe Dressings

Help improve recovery after injury!



TOTAL PATIENT CARE
PTY. LTD.

Telephone 02 9878 6188 or Fax 02 9878 6795
Email : sales@totalpatientcare.com.au
Website :www.totalpatientcare.com.au

